

PRODUCT INSTRUCTION SHEET

REFERENCE

Birstall Bariatric Mobile Commode (195)

DESCRIPTION

A robust dip-coated (non-peel) all-welded frame with a cellular moulded polyurethane backrest and over seat. Features include a clip-on/clip-off moulded seat and 14-litre pan with a lid. Fixed height and detachable arms for lateral transfer. Fitted with 4 x 125mm diameter braked castors.

The Birstall is an aid for able bodied people to transport users/occupants quickly and safely between locations and to allow for easy manoeuvrability of the chair when unoccupied.

TECHNICAL SPECIFICATIONS

Seat height (top of commode seat):	480mm (19")
Width between arms:	620mm (24½")
Seat depth:	440mm (17½")
Seat to top of arms:	200mm (8")
Seat to top of back:	440mm (17½")
Aperture width:	250mm (10")
Aperture depth:	280mm (11¼")
Footprint width:	630mm (25")
Footprint depth:	550mm (21½")
Potty diameter:	275mm (11")
Maximum user weight limit:	203kg (32st)



SAFETY USAGE

- Ensure castors are fully secure before and after use. Castor security must be checked periodically. Appliances to be used on a level surface only.
- Do not exceed weight limit and ensure that patients do not 'swing-on' the frames either in a forward or backward motion. The 4 castors must be in contact with the floor at all times.
- When issuing a shower chair a risk assessment should be carried out as to the suitability of the equipment for the user, with particular emphasis on the possibility of entrapment
- When in use, all four brake castors should be secured at all times. The front two castors should have the brake pedal facing forward, and parallel with the side of the chair. This will prevent the chair from tipping if the client leans forward.
- Ensure that detachable side arms are placed securely into the tube inserts on either side of the toilet commode seat, and secured with the pin (e-clips) provided.

PRODUCT INSTRUCTION SHEET

CLEANING AND CARE

- This product is intended for use in a wet room or shower.
- It is not intended for use outside and should not be immersed in water for long periods of time.
- The chair is designed to withstand commercial washing processes up to 80°C. Alternatively, the chair can be cleaned with a soft cloth, warm water and mild detergent.
- Ensure the product is dried thoroughly after use, or after being cleaned.
- Do not use bleach-based products on the polyurethane upholstery, as this can lead to discolouration. Chlorine solution up to 1000 ppm (parts per million) may be used on the frame only.

WARRANTY

Your Drive DeVilbiss Healthcare branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment.

This warranty is an expression of our confidence in the materials and workmanship of our products, and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist. This does not affect your statutory rights.