

PRODUCT INSTRUCTION SHEET

REFERENCE

Bariatric Equipment Extra Wide Perching Stool
Codes 402B/W ~ 403B/W ~ 404B/W ~ 405B/W ~ 406B/W

DESCRIPTION

Extra wide perching stool with PVC heat sealed upholstery. High tensile steel tubing, all welded to maximum user weight of 280kg (44 Stones) Upholstery available in brown or white – Various Models available

TECHNICAL SPECIFICATIONS

Seat Adj Height:	18" – 24"
Width between Arms:	25"
Seat Width x Depth:	20" x 12"
Product weight	6kg
Weight capacity	280kg (44st)



USAGE REQUIREMENTS

1. Use on level surfaces only, do not exceed user weight of 280kg (44 Stones)
2. Where height adjustment is required ensure that there is no gap between the horse shoe clip and frame. The clip spindle must be fully located right through the outside frame.
3. Ensure users do not "swing-on" the frames either in a forward or backward motion. All four legs of the unit must be adjusted to the same height, and be in contact with the floor at all times.
4. Periodically check the horse shoe clip locating hole to ensure that it has not become elongated or stressed.
5. Check for split, worn or loose ferrules. Check that all upholstery is fully secured at all times.
6. Designed for minimum maintenance-clean with a soft cloth using general purpose neutral detergent and warm water. Do not use harsh abrasives. Disinfect with wipe over solution of 1000 parts per million chlorine disinfectant (haztabs/precept), rinse and dry.