

PRODUCT INSTRUCTION SHEET

REFERENCE

Perching Stool 507B / 507W
Perching Stool (707PU)

DESCRIPTION


Practical perching stool manufactured in high Grade with a tough nylon coating. All models Have front to back struts for additional strength. The seats 13½” x 11½” (343mm x 292mm) are padded and angled for improved posture and have an option of a PVC heat sealed seat or a PU (polyurethane) seat.

TECHNICAL SPECIFICATIONS

Adjustable height (to seat top):	21” – 27” (54 – 69cm)
Width x depth	23” x 16” (58 x 41cm)
Seat size:	13” x 11” (33 x 28cm)
Back size:	10 ½” x 6” (27cm x 15cm)

USAGE REQUIREMENTS

1. Do not exceed recommend weight (165kg-26 Stone)
2. Before, during and after use, ensure that seat fixing screws are secure.
3. The product is not designed to be “swung on” either in a backward, sideways or forward motion. The 4 ferrules must be in contact with the ground at all times. Use on level surfaces only.
4. When adjusting the legs, ensure that there is no gap between horseshoe clip and frame. The clip spindle must be fully located right through the outside frame.
5. Clean frame with cream cleaner, upholstery with soap and water. Ensure there is no damage to frame coating and check there are no gaps on upholstery welds.

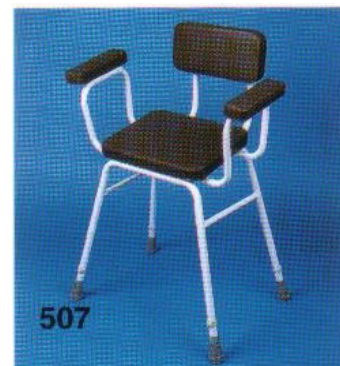

507

