

Warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 year warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

Drive DeVilbiss Healthcare Ltd

Sidhil Business Park, Holmfield, Halifax,
West Yorkshire, HX2 9TN, Great Britain

info@drivedevilbiss.co.uk
www.drivedevilbiss.co.uk

Tel: +44 (0) 845 0600 333

Drive DeVilbiss Sidhil Ltd

4 Trench Road, Mallusk, Newtownabbey,
BT36 4TY, Northern Ireland

drive

DeVilbiss
HEALTHCARE



Bath Bench

User Manual

drive | **DeVilbiss**
HEALTHCARE

Safety Precautions

Ensure that all the push buttons lock into position and protrude through the holes in the legs.

Check that all the fastenings are tightened.

Check the rubber tips for rips, wear & tear, replace if necessary.

Do not stand on the bench.

Always sit in the centre of the bench, do not lean on the edges of the bench as this can cause instability.


Do not exceed the maximum weight limit of 135kg (21 stone).

The bench is only suitable for use by one user at a time.

Assembly

Inspect all parts for damage before assembly. If there is any damage - DO NOT USE. Contact your stockist for further instructions.

1. Lay the seat upside down on a flat surface, so that the 4 leg receptacles are facing upwards.
2. Fully insert the first leg into the leg receptacle. Rotate the leg until it 'clicks' into position.
3. Repeat for the other 3 legs, ensuring that they have all 'clicked' into position.

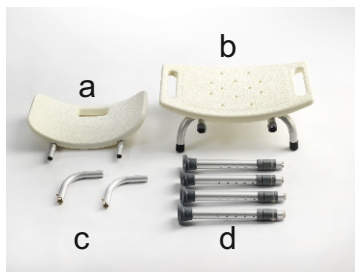


(a) Backrest
(b) Seat
(c) Back Support Tubes
(d) Legs

Models 12002KD & 12002KDR are supplied with a backrest.

First attach the back support tubes (c) to the backrest (a). Ensure that the push buttons snap out through the holes.

Then attach the backrest to the seat by inserting both back support tubes into the back support receptacles on the seat. Ensure that the push buttons snap out and lock the backrest in place.

Height adjustment

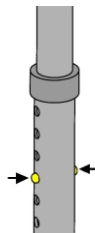

To adjust the leg height:

1. Push in the buttons (as shown).
2. Adjust the section to the required height.
3. When this height is reached, release the button but carry on adjusting the section until the buttons lock in position.
4. Repeat for the other 3 legs.

Caution! Ensure that all 4 legs are set to the same height.

Specification

The bath bench is constructed from aluminium tubing, PE-HD white plastic seat and rubber tips.

Product Code:	12002KD	12002KDR	12003KDR
With Backrest	Yes	Yes	No
Max user mass	135kg (21st)		
Overall D x W	53cm x 50cm		
Overall Height	82cm	82cm	40-50cm
Seat D x W x H	30 x 50 x 40-50cm		

Maintenance

To maintain the quality of finish, we recommend that the bath bench is wiped down and dried after each use. Periodically, clean the stool with a damp cloth and mild detergent or disinfectant. For example, chloride disinfectant (solution of 1000 parts per million) can be used.

Check for any damage to the seat and/or legs, if in doubt, remove from service.