

INSTRUC/CASA/ELITE Addendum January 2021

This document is to accompany any bed sold from January 2021, with revision 0 of the Casa Elite IFU, in order to highlight several changes in the Casa Elite instructions for use.

Section 3.6 EN 60601-2-52

The Casa Elite instructions for use state that the Casa Elite Low bed does not meet the upper height requirements for this standard, it also does not meet the under bed requirements due to the low height nature of the bed.


Section 6. Symbol Definitions

The Casa Elite bed instructions for use currently display a symbol showing items with a weight of 20kg or more. This has been removed from the bed.

Section 8. Removal from Transport Stands

The image shown in the instructions for use under 8.1.1 should be ignored, as it has been obsoleted by the new image below.

OLD IMAGE - TO BE IGNORED

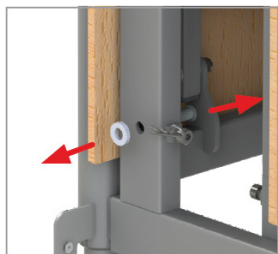

8.1.1 Clevis pin, spacer, and 'R' clip removal


NEW IMAGE - TO BE REFERENCED



8.1.1 Clevis pin, spacer, and 'R' clip removal


The image shown in the instructions for use under 8.2.1 should be ignored, as it has been obsoleted by the new image below. Referring to the old image could result in product damage.

OLD IMAGE - TO BE IGNORED

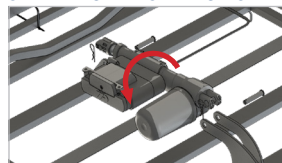

8.2.1 Rotation of backrest actuator

NEW IMAGE - TO BE REFERENCED

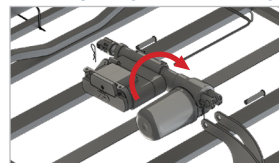

8.2.1 Rotation of backrest actuator

Section 8.10 Fitting of Ratchet

It is stated in section 8.10 that the Casa Elite is provided with a ratchet, however this has been updated and the ratchet is now available as an accessory. For more information see Section 18 of the Casa Elite instructions for use.

Section 14.2 Fault Finding

If the electrical system is working slowly, and the bed has the CASA/BATTERY battery backup accessory, the bed may be operating solely on battery power. Ensure the bed is plugged into the mains and turned on (the power light on the control box will be visible).

Section 19. Service Life

The Casa Elite instructions for use currently state Section 19 as the warranty page, however this section has been added to the instructions for use, and the warranty section will become section 20 in future versions of the instructions for use document. The service life of the Casa Elite and Casa Elite Low beds is 10 years*, with the exception of: CASA/BATTERY battery backup accessory, side rail pads and mattresses.

* On the basis that the bed and its associated accessories are serviced and maintained in accordance with the information detailed in section 14 of the Casa Elite instructions for use and the individual instructions provided with the accessory in question.



drive

DeVilbiss
HEALTHCARE

Drive DeVilbiss Healthcare Ltd,
Sidhil Business Park, Holmfild,
Halifax, West Yorkshire, HX2 9TN
United Kingdom

Tel: + 44 (0) 845 0600 333

Fax: + 44 (0) 845 0600 334

Email: sales@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk