

Head & Foot End Pads



Instructions for use

www.sidhil.com

INTRODUCTION

Thank you for purchasing the Solite Pro head and foot end pads. These pads are only designed for use with the Solite Pro range of beds; please refer to your bed instructions for use or contact your provider / supplier for further details. These instructions for use cover the fitting procedure and should be fully read and understood before the product is used.

PART IDENTIFICATION

Package contents:

Head and foot end pad x 2

Each pad features:

1. Elastic strap x 2
2. Upper strap x 2
3. Lower strap x 2



FITTING THE PADS




Warning

Before the pads are fitted to a bed, a risk assessment must be performed on a patient by patient basis to ensure suitability.

The elastic straps on the head and foot end pads will adjust to the width of the bed end depending on whether side rail channels are fitted.

Slot the pad over the bed end with the padded side facing inwards, ensuring that the elastic straps are positioned on the outside of the side rail channels (if fitted). Feed the lower straps under the head / foot board and secure to the upper straps around the back of the bed end.

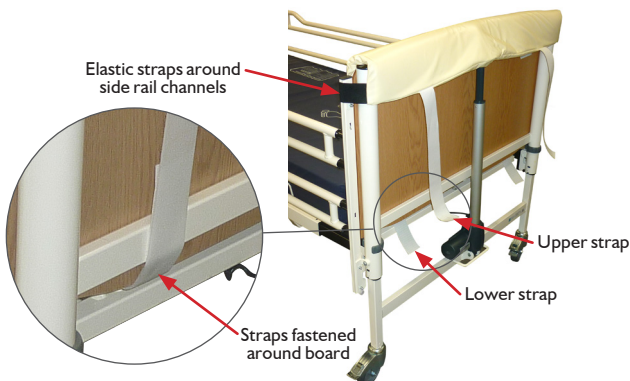

Caution

Ensure the pads are correctly strapped to the bed ends. Loose or incorrectly strapped pads may cause damage to the bedframe.

MAINTENANCE

Sidhil recommends that the carer performs frequent visual inspections. If there are any signs of contamination or damage, the affected pad is to be withdrawn from use and replaced.

Periodically check to ensure that;

- There are no signs of tearing, punctures or abrasion marks to the pads.
- There are no signs of splitting or tearing of the cover seams.
- There are no signs of staining, dampness or smell for confirmation of fluid ingress.
- The product is cleaned following the guidelines in these instructions for use.

CLEANING



Warning

- Regular cleaning and disinfection of the pads will help to prevent the risk of infection to the occupant and / or carer.
- Prior to transferring the pads to another user ensure they have been cleaned and disinfected using the method as detailed below to help prevent the risk of cross infection.

General Cleaning:

- The pads should be cleaned by starting with the cleanest parts and systematically moving to the dirtiest parts. Extra care should be taken around areas where excess dirt or dust may gather.
- The cloth should be changed during the cleaning process if it becomes soiled.
- Wipe down with a clean cloth moistened with a mild detergent and dilute with warm water (40°C).
- Rinse with cold, clean water and a clean cloth, and allow to fully dry before use.

Decontamination:

Refer to the bed's instructions for use for full decontamination guidelines.

SPECIFICATION

Product code	SOLITE/PRO/HF/PAD
Height (mm)	560
Width (mm)	990
Foam thickness (mm)	20
Fire resistance	BS 7176:2007 - Low hazard

WARRANTY

Sidhil Ltd. warrants that this product will perform in accordance with its specification and will remain free from defects in material and workmanship when used under normal conditions for a period of 1 year (1 year full parts and labour) from the date of purchase from Sidhil Ltd. and its subsidiary companies. If purchased from an authorised dealer or international distributor, the product is warranted for 1 year parts only. SIDHIL LTD. MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ALL IMPLIED WARRANTIES OF MERCHANTABILITY, NON-INFRINGEMENT AND FITNESS FOR A PARTICULAR PURPOSE ARE HEREBY DISCLAIMED. IN NO EVENT WILL SIDHIL LTD. BE LIABLE FOR PUNITIVE, SPECIAL OR CONSEQUENTIAL DAMAGES, OR FOR AN AMOUNT IN EXCESS OF THE PURCHASE PRICE OF THE DEFECTIVE SIDHIL LTD. PRODUCT OR PRODUCTS.

This limited 1 year warranty gives you specific legal rights and you may also have other rights.

All products carry the CE mark in accordance with EC Directive on Medical Devices (93/42/EEC).

Sidhil Ltd. has a policy of continual product improvement and reserves the right to amend specifications covered in this document.

Please contact Sidhil Ltd. for complete warranty information.

No part of this document may be reproduced without the written approval of Sidhil Ltd.



CONTACT INFORMATION

Tel: +44 (0) 1422 233000 Fax: +44 (0) 1422 233010

Email: sales@sidhil.com www.sidhil.com

Sidhil Business Park, Holmfield, Halifax, HX2 9TN

A member of the Siddall & Hilton Ltd. Group of Companies



Certificate No. FM14550

INSTRUC/SOLITE/HF/PAD, 05/04/16 - Rev.0