

Standard Length Integral Side Rail Pads



Integral Side Rail Pad Extensions



Instructions for use

CONTENTS

- 1. INTRODUCTION 1
- 2. CONTACT INFORMATION 1
- 3. PART IDENTIFICATION 2
 - 3.1 Standard Length Integral Side Rail Pads 2
 - 3.2 Integral Side Rail Pad Extensions..... 2
- 4. FITTING THE STANDARD LENGTH INTEGRAL SIDE RAIL PADS..... 3
- 5. FITTING THE INTEGRAL SIDE RAIL PAD EXTENSIONS 4
- 6. MAINTENANCE 5
- 7. CLEANING..... 5
- 8. SPECIFICATION 5
- 9. WARRANTY 6

I. INTRODUCTION

Thank you for purchasing the Solite Pro standard length integral side rail pads / integral side rail pad extensions. These pads are only designed for use with the Solite Pro range of beds fitted with integral side rails; please refer to your bed instructions for use or contact your provider / supplier for further details. These instructions for use cover the fitting procedure of both standard length integral side rail pads and integral side rail pad extensions. As a result, some fitting procedures detailed in this document may not apply to your product; refer to the contents page to identify the sections relevant to your product. Please ensure that you understand all instructions.

2. CONTACT INFORMATION

For any service, warranty, sales or customer service information regarding this product please contact your provider or if in doubt contact Sidhil Ltd. at the following address:

Sidhil Ltd.
Sidhil Business Park
Holmfield
Halifax
West Yorkshire
HX2 9TN
United Kingdom

Please quote the product code and date of manufacture on all correspondence. This can be found on the identification label stitched to each pad.

Service & Maintenance

Tel: +44 (0)1422 233136

Fax: +44 (0)1422 233010

Spares

Tel: +44 (0)1422 233138

Fax: +44 (0)1422 233010

Customer Service

Tel: +44 (0)1422 233000

Fax: +44 (0)1422 233010

e: sales@sidhil.com

w: www.sidhil.com

For Service & Support outside the United Kingdom & Northern Ireland please contact the local distribution company from where this equipment was purchased. Failure to do so may result in the manufacturer's warranty becoming void.

3. PART IDENTIFICATION

Your package contents will vary depending on the product you have purchased; standard length integral side rail pads or integral side rail pad extensions.

3.1 Standard Length Integral Side Rail Pads

Package contents:

Standard length integral side rail pad x 2

Each pad features:

1. Side rail pad mesh section x 1
2. Inner fastening strip x 2
3. Pad connecting strip x 2
4. Outer fastening strip x 2




3.2 Integral Side Rail Pad Extensions

Package contents:

Integral side rail pad extension x 2

Each pad features:

1. Inner fastening strip x 2
2. Outer fastening strip x 2
3. Pad connecting strip x 1

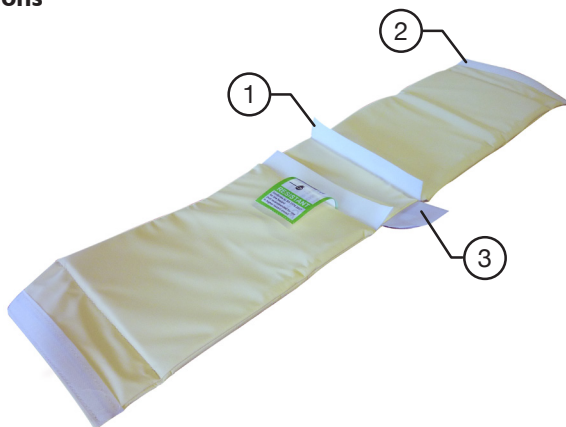

4. FITTING THE STANDARD LENGTH INTEGRAL SIDE RAIL PADS



Warning

Before the pads are fitted to a bed, a risk assessment must be performed on a patient by patient basis to ensure suitability.

Prior to fitting the standard length integral side rail pads, the bed must be fitted with standard length integral side rails. If in doubt please contact your provider or Sidhil Ltd. for further advice. Ensure the side rails are in the raised position and the mattress is removed from the bed (if fitted) prior to performing the assembly procedure below.

1. Lay the pads flat on a clean smooth surface and open all fastening strips.
2. Whilst ensuring one end of the side rail pad is positioned up to the head board, wrap the side rail pad around the side rails from the inside of the bed, and secure the fastening strips together on the outside of the bed.



4.1 Pad laid flat



4.2 Fasten the pad around the side rails

3. Check all fastening strips are secured correctly around the side rails.
4. Raise and lower the side rails to ensure the movement is not restricted. Repeat for the remaining side rail pad.



4.3 Check all fastening strips are correctly secured



4.4 Test the movement of the side rail



Caution

Ensure the pads are correctly strapped to the side rails. Loose or incorrectly fastened pads may cause damage to the side rails.

5. FITTING THE INTEGRAL SIDE RAIL PAD EXTENSIONS



Warning


Before the pads are fitted to the side rails, a risk assessment must be performed on a patient by patient basis to ensure suitability.

Prior to fitting the side rail extension pads, the bed must be fitted with extended length integral side rails and standard length integral side rail pads. If in doubt please contact your provider or Sidhil Ltd. for further advice. Ensure the side rails are in the raised position and the standard length integral side rail pads are positioned up to the head board before attempting to fit the integral side rail pad extensions.

1. Lay the side rail pad extensions flat on a clean smooth surface and open all fastening strips.
2. Wrap the side rail pad extension around the foot end of the side rails from the inside of the bed, and secure the fastening strips together on the outside of the bed.




5.1 Side rail pad extension laid flat

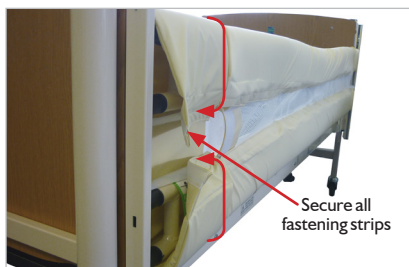

5.2 Fasten the pad around the side rails

3. Fasten the pad connecting strips on the side rail pad and the side rail pad extension together. Check all fastenings strips are secured correctly around the side rails.
4. Raise and lower the side rails to ensure the movement is not restricted. Repeat for the remaining side rail pad.

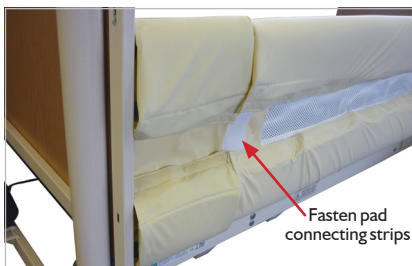

5.3 Fasten pad connecting strips



5.4 Testing movement of the side rail



Caution

Ensure the pads are correctly strapped to the side rails. Loose or incorrectly strapped pads may cause damage to the side rails.

6. MAINTENANCE

Sidhil recommends that the carer performs frequent visual inspections. If there are any signs of contamination or damage, the affected pad is to be withdrawn from use and replaced.

Periodically check to ensure that;

- There are no signs of tearing, punctures or abrasion marks to the pads or mesh.
- There are no signs of splitting or tearing of the cover seams.
- There are no signs of staining, dampness or smell for confirmation of fluid ingress.
- The product is cleaned following the guidelines in these instructions for use.

7. CLEANING



- Regular cleaning and disinfection of the pads will help to prevent the risk of infection to the occupant and / or carer.
- Prior to transferring the pads to another user ensure they have been cleaned and disinfected using the method as detailed below to help prevent the risk of cross infection.

General Cleaning:

- The pads should be cleaned by starting with the cleanest parts and systematically moving to the dirtiest parts. Extra care should be taken around areas where excess dirt or dust may gather.
- The cloth should be changed during the cleaning process if it becomes soiled.
- Wipe down with a clean cloth moistened with a mild detergent and dilute with warm water (40°C).
- Rinse with cold, clean water and a clean cloth, and allow to fully dry before use.

Decontamination:

Refer to the bed's instructions for use for full decontamination guidelines.

8. SPECIFICATION

Product code	SOLITE/PRO/SR/PAD	SOLITE/PRO/SR/EXT/PAD
Unfolded height (mm)	760	760
Unfolded width (mm)	1990	180
Foam thickness (mm)	15	15
Fire resistance	BS 7176:2007 - Low hazard	BS 7176:2007 - Low hazard

9. WARRANTY

Sidhil Ltd. warrants that this product will perform in accordance with its specification and will remain free from defects in material and workmanship when used under normal conditions for a period of 1 year (1 year full parts and labour) from the date of purchase from Sidhil Ltd. and its subsidiary companies. If purchased from an authorised dealer or international distributor, the product is warranted for 1 year parts only. SIDHIL LTD. MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ALL IMPLIED WARRANTIES OF MERCHANTABILITY, NON-INFRINGEMENT AND FITNESS FOR A PARTICULAR PURPOSE ARE HEREBY DISCLAIMED. IN NO EVENT WILL SIDHIL LTD. BE LIABLE FOR PUNITIVE, SPECIAL OR CONSEQUENTIAL DAMAGES, OR FOR AN AMOUNT IN EXCESS OF THE PURCHASE PRICE OF THE DEFECTIVE SIDHIL LTD. PRODUCT OR PRODUCTS.

This limited 1 year warranty gives you specific legal rights and you may also have other rights.

All products carry the CE mark in accordance with EC Directive on Medical Devices (93/42/EEC).

Sidhil Ltd. has a policy of continual product improvement and reserves the right to amend specifications covered in this document.

Please contact Sidhil Ltd. for complete warranty information.

No part of this document may be reproduced without the written approval of Sidhil Ltd.



CONTACT INFORMATION

Tel: +44 (0) 1422 233000 Fax: +44 (0) 1422 233010

Email: sales@sidhil.com www.sidhil.com

Sidhil Business Park, Holmfield, Halifax, HX2 9TN

A member of the Siddall & Hilton Ltd. Group of Companies



Certificate No. FM14550

INSTRUC/SOLITE/PRO/SR/PAD, 05/04/16 - Rev.0