

XIAMEN ORTOCARE SPORTS HEALTH CO., LTD

SUNGO Cert GmbH Harffstr. 47,40591 Düsseldorf, Germany

5 years Disposal: Please dispose the product after use to comply with local regulation. Notice to

Knee Brace Intended Use: Knee Brace Helps immobilize and provide support on knee joint.

Usage: 1. Measure circumference of thigh. 2. Refer to sizing chart to select proper size. 3. Grasp side of sleeve at top, which is indicated by logo and larger opening. Step into sleeve and pull up so that the product surrounds the thigh area.

Cleaning Methods / Maintenance: Hand wash in cold water with mild soap. Dry clean, bleach or tumble is prohibited. Natural dry, and do not expose this brace to any heat source. Precaution and Warning:

Please consult with your physician if any allergic symptom occurs.

Do not use more than 24 hours of continuous.

Storage: Store the product in a cool dry place away from sunlight and heat