

BRAUN

Manual Nasal Aspirator 1

Certain trademarks used under license from
The Procter & Gamble Company or its affiliates.



 Kaz USA, Inc., A Helen of Troy Company
400 Donald Lynch Boulevard
Suite 300 Marlborough
MA USA 01752

Distributed by:
Felton Grimwade & Bosisto's Pty. Ltd.
61-81 Clarinda Road (PO Box 74),
Oakleigh South, Victoria 3167, Australia

Customer service:
Free call: 1800 655 841
Website: www.fgb.com.au



Australian Sponsor:
DH RegSys Consulting Pty Ltd
Level 3, 50 Pirie Street,
Adelaide, SA 5000 Australia

© 2024, All rights reserved.

www.BraunHealthcare.com



Made and printed
in China

MD REF BNA050AU
P/N: A009082R0
13MAR24

Product description

The Braun Manual Nasal Aspirator 1 is a device that is used to suction mucus from a baby's nasal passages using human suction. The device contains a mouthpiece, tube and mucus receptacle.

Intended use

The Braun Manual Nasal Aspirator 1 (and associated filters) is intended to be used for providing intermittent suction to remove nasal secretions and mucous in children, 0 to 12 years of age, for home use only. The associated filters (in this box and BNF020AU) are intended for single use only.



Please read all the instructions before using the Braun Manual Nasal Aspirator 1, especially this basic safety information.

- The Braun Manual Nasal Aspirator 1 is designed to clean/extract nasal secretions only. Do not use the device for purposes other than those stated in these instructions for use.
- Keep out of reach of children under 12 years.
- Do not expose the Braun Manual nasal aspirator 1 to direct sunlight.
- This device is designed for domestic/home use. Consult a doctor if the child is suffering from a nasal infection.
- Do not use other nozzle than those supplied with this device.
- Do not reuse filter.
- Only use Braun single-use nasal aspirator filters with this device.



CHOKING HAZARD - May contain small detachable parts. Keep out of reach of children under 3 years old.

Cleaning

To clean the Braun Manual Nasal Aspirator 1, twist off the cup and remove the chamber. Clean it with soap and warm water except the tube. The detachable part (mouth piece, nose tip and chamber) can be placed in the dishwasher except the tube. Alternatively, all parts of the device can be boiled for 15 minutes. This last method is mandatory for the tube.

Replace filter (single use device filter BNF020AU) with every use to maintain hygiene and to prevent cross-contamination.

Guarantee

Please read all instructions before attempting to use this device. Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant guarantee period. Any claim under the guarantee will not be valid without a proof of purchase. Your device is guaranteed for Two Years (2 Years) from date of purchase. This guarantee covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge. The guarantee DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The guarantee becomes void if the device is opened, tampered with, or used with non-Braun branded parts or accessories, or if repairs are undertaken by unauthorized persons. Accessories and consumables are excluded from any warranty.

REF, LOT of your product must be provided as they are essential to record and follow up your inquiry or claim.

The production date is given by the LOT number and can be deciphered as explained below:

The first 3 numeric digits in the LOT number represent the day of the year of manufacture.

The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letter(s) at the end designate the manufacturer of the product. (E.g.: LOT No.: 12313tav this product was made on the day 123, year 2013 at the manufacturer code tav.)

Product specifications

Model	Manual nasal aspirator (BNA050)
Operating temperature	+10 °C ~ +40 °C
Storage temperature	-25 °C ~ +55 °C
Operation & storage relative humidity	≤95 % R.H. non-condensing
Unit weight	37 g
Service life	5 years
Guarantee	2 years

Single-use filter specifications

Model	BNF020AU
Operating temperature	+10 °C ~ +40 °C
Storage temperature	-25 °C ~ +55 °C
Operation & storage relative humidity	≤95 % R.H. non-condensing



Caution



See instruction for use



Single use filter



Operating temperature



Storage temperature



Operating and storage humidity



Medical device



Catalogue number



Batch code



Manufacturer



Date of manufacture

Subject to change without notice.



Select nose tip for child based on nostril size



Ensure filter is in place and chamber is securely screwed together



Insert nose tip into child's nostril to create a seal



Make sure it is well inserted into the nostril



Do not insert too far



Use mouthpiece to suck out mucus



After use, dispose of filter



Disassemble and clean in sink or dishwasher, except tube, top rack only



Disassemble and boil for 15 minutes (mandatory for tube)



Place new filter in filter holder